
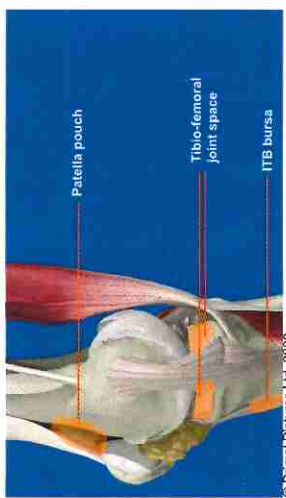


**OPERATION**

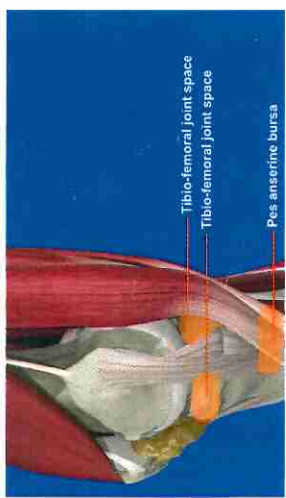


© Primal Pictures Ltd, 2000

This model contains these conductive anatomical points:

- \* Patella pouch
- \* Tibio-femoral joint space
- \* ITB bursa
- \* Pes anserine bursa

Locate the appropriate spot for injection (i.e. joint space bursa).  
Insert needle into location. The audio feedback box will emit a tone when the tip of the needle has made contact with the correct area.  
Use the dry erase marker to identify injection targets.

**Knee Conductive Injection Model  
Item # 1517-1**

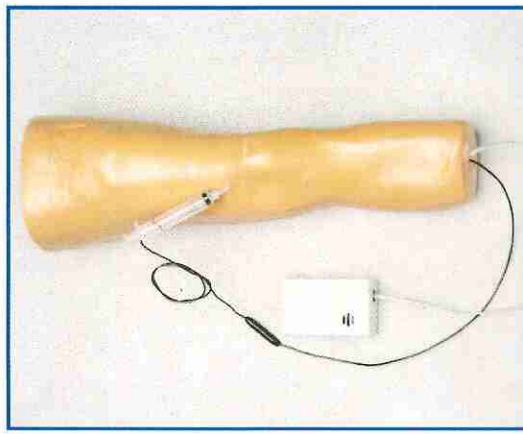

Thank you for purchasing the Sawbones conductive injection model. This injection trainer can be used to teach or demonstrate needle placement for aspiration and joint injection. With the correct needle insertion, an audible tone will sound indicating proper placement of the chosen area. Palpation of the major anatomical landmarks may also be performed. Additional conductive structures may be added upon request. With a little care and maintenance your new model should provide you with many years of service.

This instructional brochure will take you through:

- \* Components
- \* Set up instructions
- \* Operation and anatomical features
- \* Care and maintenance
- \* Available replacement parts

**SAWBONES®**

Should you have any questions, please contact:

**SAWBONES**  
10221 SW 188th Street  
Vashon, WA 98070  
(206) 463-5551  
info@sawbones.com

**SAWBONES EUROPE AB**  
Krossverksgatan 3  
216 16 Malmö  
Sweden  
+46 40 650 70 00  
info@sawbones.se  
www.sawbones.com

**SAWBONES®**