


# Ingrowing Toenail

This kit should have been supplied to you with a plastic seal around the handle. If the seal is missing please check that you have the following contents:

- CD-ROM
- Thwaites nipper
- Tourniquet
- Tourniquet clip
- Syringe
- Needle
- Nail probe
- Nail chisel
- Artery forceps



- Toe jig  
80071

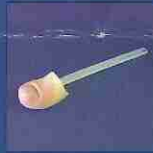

- Toe end  
80070

If an item is missing contact your local distributor, quoting part number **90051** and the **6 digit number** on the small white sticker on the side of the case.

## Replacement items

Items with **orange numbers** can be ordered from your local distributor. Distributor contact details can be found at [www.limbsandthings.com](http://www.limbsandthings.com)

## Attaching a toe end


**1** Thread the flexible plastic strip attached to the toe through the slot at the front of the spindle on the toe jig.



**2** Push the plastic strip all the way through the spindle until it appears from the opposite end.



**3** Grip the end of the strip and pull slowly until the toe end fits snugly against the front of the spindle. The toe assembly is now ready to use.

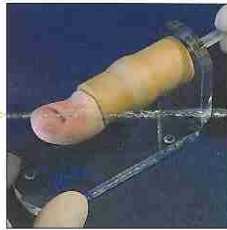

## Removing a toe end

Depress the clip at the back of the spindle, underneath the plastic strip, and gently pull the toe end until it is clear of the toe jig.

